

Embryo Safe Disinfectant for IVF Laboratory Use



Manufactured in
ISO 13485: 2016 facility

MEA & SMA TESTED



EMBRYO SAFE DISINFECTANT

for CO2 incubators, micromanipulators
and laminar flow hoods

MEA & SMA Tested

Colourless and odourless

Alcohol free, QAC free

Suitable for glass, aluminium,
stainless steel, wood, plastic,
polished and coloured surfaces

Disinfection in 30 seconds

Security seal

Non-corrosive

No need for rinsing after applying

Active Ingredient: Hypochlorous Acid which
is naturally produced by human immune system
and highly pathogenic against microorganism.

USAGE

Spray: Ready-to-use solution Embryo Safe is
sprayed on surface. It is recommended
to wait for 30 secs for the disinfection
effect.

Wipe: The surface is wiped down until it is
visibly wet and shine. It is recommended to
wait for 30 secs for the disinfection effect.

Bactericidal

Virucidal

Fungicidal



EMBRYO SAFE DISINFECTANT

for Hand

MEA & SMA Tested

Colourless and odourless

Alcohol free, QAC free

Does not cause irritation on skin

Disinfection in 30 seconds

Security seal

Non-corrosive

No need for rinsing after applying

Active Ingredient: Hypochlorous Acid which is naturally produced by human immune system and highly pathogenic against microorganism.

USAGE

Spray: Ready-to-use solution Embryo Safe is pumped on hand. It is recommended to wait for 30 secs for the disinfection effect.

Wipe: All fingers and nails as well as palms and back of both hands are wiped. It is recommended to wait for 30 secs for the disinfection effect.

Bactericidal

Virucidal

Fungicidal



EMBRYO SAFE DISINFECTANT

for IVF laboratory surface

USAGE

Ready-to-use solution Embryo Safe is poured in a container. A mop is soused in the container. Then the surface is wiped down with the mop. It is recommended to wait for 30 secs for the disinfection effect.

MEA & SMA Tested

Colourless and odourless

Alcohol free, QAC free

Suitable for glass, tiles, stainless steel, rubber, plastic, polished and coloured surfaces

Disinfection in 30 seconds

Security seal

Non-corrosive

No need for rinsing after applying


Active Ingredient: Hypochlorous Acid which is naturally produced by human immune system and highly pathogenic against microorganisms.

Bactericidal

Virucidal

Fungicidal



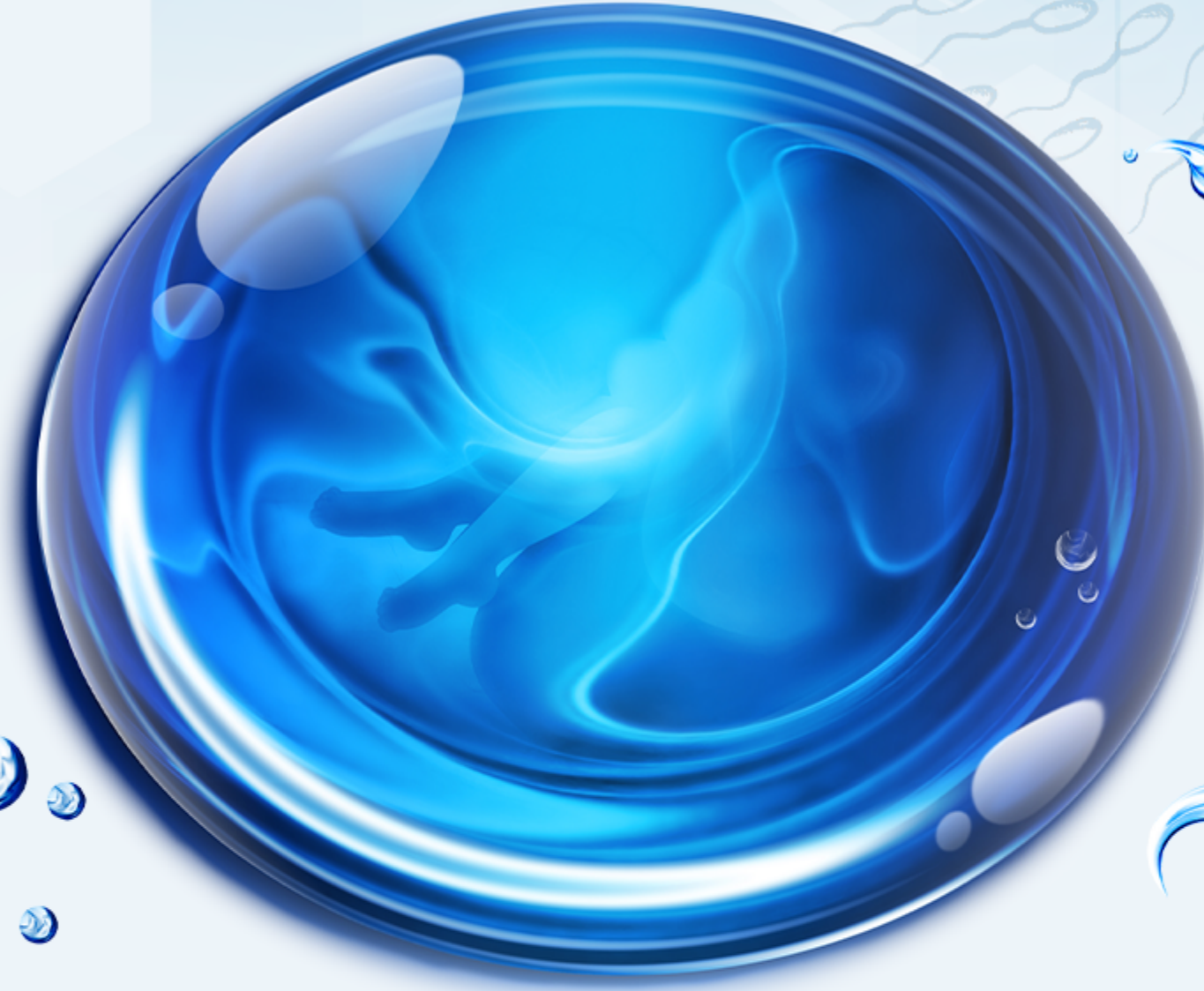

For more information, you can visit
www.ivfprime.com to find
your local dealer or send a message to
info@ivfprime.com

IVF Prime Inc.

Çamlıca Mah. Anadolu Bulv.
No: 20 Timko Sokak N1 Blok
Yenimahalle Ankara Turkey

